

MIADERMA

# Eczema Psoriasis Intensive Cream

Medical Device CE 0481

- For the treatment of dry and scaly skin caused by neurodermatitis, eczema and psoriasis\*.
- For the repair of the impaired barrier function of the skin.

\*When this has been determined by a doctor.

**General advice:** Carefully read the instructions for use prior to using Eczema Psoriasis Intensive mask, even if you used this medical device before. The information may have been amended according to new research and findings in the time since you opened your last package. This medical device can be obtained without a prescription. In spite of this, take care when using Eczema Psoriasis Intensive mask in order to obtain a good result. Keep these instructions for use in case you need to consult them again.

**Application:** Eczema Psoriasis Intensive mask is a topical medical device, which alleviates the symptoms of eczema and psoriasis, like itching, redness, scaling and enduring dry skin. Eczema and psoriasis are chronic and relapsing inflammatory diseases of the skin associated with various immunologic abnormalities.

**Dosage and use (for adults):** Eczema Psoriasis Intensive mask is designed for external use for a 4-12 weeks period, depending on initial severity. Apply to perfectly cleansed skin 2-3 times a week and massage gently into affected areas. Simultaneous use of the Dead Sea Salt Skin series products is possible.

**Duration of use:** Do not use Eczema Psoriasis Intensive mask for longer than 30 days continuously, but continued treatment might be necessary for 4-12 weeks period. Treatment can be resumed after 4 days.

**How it works:** Active ingredients are natural Dead Sea minerals, MDS<sup>17</sup> and UDS<sup>10</sup>, which have highly moisturizing properties. Dead Sea minerals also help to relieve itching and irritation. The Dead Sea minerals have moisturizing properties because of their high magnesium content, which enhances the ability of the skin to retain water. The Dead sea minerals and the MDS<sup>17</sup> are absorbed in the upper skin layer, where they reduce the loss of moisture and restore the skin barrier function. Moreover, UDS<sup>10</sup> is one of three natural moisturizing factors in the outer horny layer of the skin. It is naturally present in healthy skin.

**Possible side effect(s):** Possible side effects when using Eczema Psoriasis Intensive mask are mild skin irritation, temporary burning, stinging or itching.


**Use during pregnancy and/or breastfeeding:** No data is available about using Eczema Psoriasis Intensive mask during pregnancy or the breastfeeding period. If you are pregnant or breastfeeding, please consult your doctor or pharmacist before using this product.

 **Warnings:**

- For external use only;
- Do not use when hypersensitive or allergic to any of the ingredients;
- Keep out of reach of young children.

**Ingredients:** Aqua, MDS<sup>17</sup>, Caprylic/Capric Triglyceride, UDS<sup>10</sup>, Cetearyl Alcohol, DDS<sup>5</sup>, Glycerin, Cetearth-20, Phenoxyethanol, Argania Spinosa Kernel Oil, Tocopheryl Acetate, Ethylhexylglycerin.

**Content:** 50 ml tube.

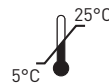

**Storage instructions:** Store at room temperature (5-25°C) in a dry and dark place in the original container and out of reach of small children.



Do not use Eczema Psoriasis Intensive mask after the date shown on the package: after "do not use after". Do not use the device more than 3 months after opening. Put the tube after use back in the original container. Be sure that the container is properly closed after use.

**Medical device:** Eczema Psoriasis Intensive mask is a class IIa medical device in accordance with the European Directive for medical devices 93/42/EEC.

Date on which these instructions for use were last revised: October 2019 • 080468V1.5



PK Benelux BV  
Vluchtoord 17, 5406 XP Uden, NL



If you would like further information about this product, please contact:

Miaderma Regulatory Services  
Samuel Ryder House, Barling Way  
Nuneaton, Warwickshire, CV10 7RH  
United Kingdom

**Telephone:** +44 (0) 2476 215 400

**Fax:** +44 (0) 2476 215 452

**Email:** [customerservices@hollandandbarrett.com](mailto:customerservices@hollandandbarrett.com)

CE 0481

00A